

Note: The most current version of these instructions can be found online at www.grahamfield.com

- ⚠ **WARNING: Important! Read and understand these instructions before installing or using the Toilet Seat Rail. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If this product is not properly assembled and installed, personal injury and damage could result.**
- ⚠ **WARNING: If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts. Use only Lumex replacement parts. Non-Lumex replacement parts could cause personal injury and damage.**
- ⚠ **WARNING: This product has a weight limit of 250 lb.**
- ⚠ **WARNING: Users who have limited physical capabilities must be assisted or supervised when using this product.**
- ⚠ **WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.**

INTENDED USE

The Toilet Seat Rail is intended to provide support and stability during toileting.


INSTALLATION

Unpacking

Remove Toilet Seat Rail from carton and protective packaging. Check for obvious damage to the carton or its contents. If damage is evident, do not use the product; please notify the carrier and your Graham-Field authorized distributor for further instruction.

Installation (Refer to Illustration)

1. Remove existing toilet seat.
2. Place mounting bracket on flat surface of bowl and align holes. Ensure that mounting bracket tube is to the rear, closest to the tank.
3. Replace toilet seat, inserting bolts through mounting bracket holes and into toilet seat mounting holes. Replace original nuts and tighten.
4. Depress push buttons and install arms in mounting bracket as shown. Ensure the Toilet Seat Rail sleeves rest securely on the toilet rim.
5. Adjust Toilet Seat Rail arm width by engaging push buttons in desired holes in mounting bracket tube. Adjust the arms to desired width of either 17" (inner holes) or 19" (outer holes).

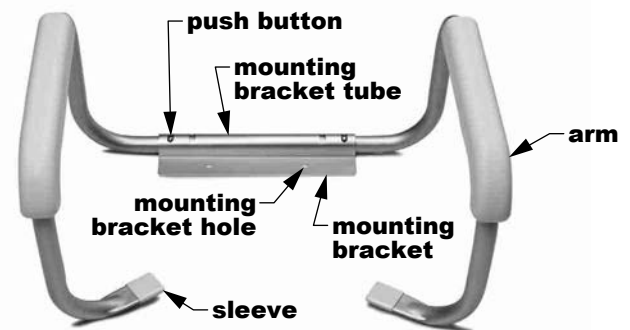

**Toilet Seat Rail,
Width-Adjustable**

- ⚠ **WARNING: Ensure that mounting bracket push buttons are locked in place before use.**

OPERATION

⚠ WARNING: Always ensure this product is securely installed before use.

CLEANING AND MAINTENANCE

Cleaning

To clean this product, wash it with soap and water or a mild detergent, then rinse it with water.

▲ NOTICE: Never use abrasive cleaners or scouring pads on any part of the Toilet Seat Rail.

Maintenance

⚠ WARNING: Inspect this product weekly and tighten it as needed to ensure secure installation. Check arms and sleeves for tears, rips or wear. If any of these conditions exist, replace arms or sleeves IMMEDIATELY.

WARRANTY

GF Health Products, Inc. warrants the Lumex 6450R-2 Toilet Seat Rail as follows:

Limited lifetime warranty against manufacturer's defects. If a product is deemed to be under warranty, GF Health Products, Inc. shall provide, at its option, (1) replacement of any defective part or product or (2) a credit of the original selling price made to GF Health Products, Inc.'s initial customer. The warranty does not include any labor charges incurred in replacement part(s) installation or any associated freight or shipping charges to GF Health Products, Inc.

The warranties contained herein contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.



GRAHAM-FIELD
www.grahamfield.com

GF Health Products, Inc. is not responsible for typographical errors. Packaging, warranties, products and specifications are subject to change without notice.

Graham-Field and Lumex are registered trademarks of GF Health Products, Inc.

© 2011 GF Health Products, Inc. • 2935 Northeast Parkway • Atlanta, GA 30360 • tel 770-368-4700